


IMPORTANT

USER INSTRUCTIONS VERSAPAK DOPING CONTROL KIT *PARTIAL URINE COLLECTION*

INTRODUCTION

User instructions for the Partial Urine Collection Kit.

 Please read all instructions carefully.

 www.versapak-anti-doping.com

INDICATIONS FOR USE

- The Versapak Doping Control Partial Urine Collection kit is to be used as part of the Anti Doping Process as per the 'International Standards for Testing and Investigations (ISTI)'.
• The Versapak Doping Control kit for Partial Urine Collection is to be used in accordance with this user instruction manual.

CONTENTS

1. A Single Urine Collection Vessel with laser engraved gradient markings and a unique number
2. A secure leak proof bag
3. A Barcode with matching number

The vessel is in one part with the cup and the lid attached.
The kit is shrink wrapped for protection.



GETTING STARTED

- Unwrap the Collection Vessel and use this for the partial urine sample according to Annex D of the ISTI.
- Close the lid of the Urine Collection Vessel by firmly pressing down on the lid with the palm of your hand until you clearly hear the closing CLICK. Ensure the collection vessel is on a flat surface when closing.
- Attach the barcode to the leak proof bag, checking the numbers match with the Collection Vessel.
- Put the closed collection vessel in the leak proof bag and seal. Keep upright until a full sample is available.
- To pour the urine into the Doping Control A and B bottles, use the ring pull to open the spout.
- Fill the A and B bottles to the levels required.



versapak[®]
Doping Control Limited

Versapak Doping Control Ltd. 4 Veridion Way, Erith, Kent DA18 4AL

Tel: +44 (0)20 8333 5320

Email: info@versapak-anti-doping.com

versapak-anti-doping.com